



User's Manual

Mumps virus (Parotitis) IgM ELISA



DEIA365R



96T



This product is for research use only and is not intended for diagnostic use.

For illustrative purposes only. To perform the assay the instructions for use provided with the kit have to be used.

Creative Diagnostics

 **Address: 45-1 Ramsey Road, Shirley, NY 11967, USA**

 **Tel: 1-631-624-4882 (USA) 44-161-818-6441 (Europe)**  **Fax: 1-631-938-8221**

 **Email: info@creative-diagnostics.com**  **Web: www.creative-diagnostics.com**

PRODUCT INFORMATION

Intended Use

The Mumps virus (Parotitis) IgM ELISA is intended for the qualitative determination of IgM class antibodies against Mumps Virus in human serum or plasma (citrate, heparin).

General Description

Mumps viruses are RNA viruses of the family Paramyxoviridae. The virions are spherical particles of 150-250 nm in diameter consisting of a ribonucleoprotein with helical symmetry and enveloped by matrix protein and a lipid bilayer which contains two spikeline structures: viral hemagglutinin (H) and viral neuraminidase (N). Mumps virus involves primarily the parotid and related salivary glands; however infection can lead to CNS disease and accumulation of the virus in CSF. Mumps (Epidemic Parotitis) is an acute contagious viral disease mostly occurring in children. Nearly 50% of all infections are subclinical. The highest incidence of clinical manifestations is found in the age group of 4 to 15 years. Secondary infections are rare because of longlasting immunity. 10 to 35 % of mumps cases develop orchitis which occurs nearly always after puberty. The process is mostly unilateral and the prognosis usually good. Mumps virus has been one of the most important causes of viral CNS disease (meningitis and encephalitis) in USA; vaccine administration has greatly reduced its incidence.

Species	Disease	Symptoms (e.g.)	Transmission route
Mumps Virus	Mumps	Fever and unilateral or bilateral swelling of the parotid gland; the sublingual and submaxillary glands may also be involved Complications: Orchitis, Meningoencephalitis, Pancreatitis	Virus transmission occurs by droplet infection

Principles of Testing

The qualitative immunoenzymatic determination of specific antibodies is based on the ELISA (Enzyme-linked Immunosorbent Assay) technique.

Microtiterplates are coated with specific antigens to bind corresponding antibodies of the sample. After washing the wells to remove all unbound sample material a horseradish peroxidase (HRP) labelled conjugate is added. This conjugate binds to the captured antibodies. In a second washing step unbound conjugate is removed. The immune complex formed by the bound conjugate is visualized by adding Tetramethylbenzidine (TMB) substrate which gives a blue reaction product.

The intensity of this product is proportional to the amount of specific antibodies in the sample. Sulphuric acid is added to stop the reaction. This produces a yellow endpoint colour. Absorbance at 450/620 nm is read using an ELISA Microtiterplate reader.

Reagents And Materials Provided

Microtiter Plate: 12 break-apart 8-well snap-off strips coated with Mumps Virus antigens; in resealable aluminium foil.

Conjugate: 1 bottle containing 20 mL of peroxidase labelled antibody to human IgM in phosphate buffer (10 mM); coloured red; ready to use; black cap.

Positive Control: 1 vial containing 2 mL control; coloured yellow; ready to use; red cap; $\leq 0.02\%$ (v/v) MIT.

Negative Control: 1 vial containing 2 mL control; coloured yellow; ready to use; blue cap; $\leq 0.0015\%$ (v/v) CMIT/ MIT (3:1).

Cut-off Control: 1 vial containing 3 mL control; coloured yellow; ready to use; green cap; $\leq 0.02\%$ (v/v) MIT.

Sample Diluent: 1 bottle containing 100 mL of phosphate buffer (10 mM) for sample dilution; pH 7.2 ± 0.2 ; anti-human IgG (RF Absorbent); coloured green; ready to use; white cap; $\leq 0.0015\%$ (v/v) CMIT/ MIT (3:1).

TMB Substrate Solution: 1 bottle containing 15 mL 3,3',5,5'-tetramethylbenzidine (TMB), $< 0.1\%$; ready to use; yellow cap.

Washing Solution (20x conc) : 1 bottle containing 50 mL of a 20-fold concentrated phosphate buffer (0.2 M), pH 7.2 ± 0.2 , for washing the wells; white cap.

Stop Solution: 1 bottle containing 15 mL sulphuric acid, 0.2 mol/L; ready to use; red cap.

Materials Required But Not Supplied

1. ELISA Microtiterplate reader, equipped for the measurement of absorbance at 450/620 nm
2. Incubator 37°C
3. Manual or automatic equipment for rinsing Microtiterplates
4. Pipettes to deliver volumes between 10 and 1000 μL
5. Vortex tube mixer
6. Distilled water
7. Disposable tubes

Storage

Store the kit at 2 - 8°C. The opened reagents are stable up to the expiry date stated on the label when stored at 2 - 8°C.

Specimen Collection And Preparation

1. Use human serum or plasma (citrate, heparin) samples with this assay. If the assay is performed within 5 days after sample collection, the samples should be kept at 2 - 8°C; otherwise they should be aliquoted and stored deep-frozen (-70...-20°C). If samples are stored frozen, mix thawed samples well before testing. Avoid repeated freezing and thawing.
2. Heat inactivation of samples is not recommended.
3. Sample Dilution: Before assaying, all samples should be diluted 1+100 with Sample Dilution. Dispense 10 μL sample and 1 mL Sample Dilution into tubes to obtain a 1+100 dilution and thoroughly mix with a Vortex.

Reagent Preparation

It is very important to bring all reagents and samples to room temperature (20 - 25°C) and mix them before starting the test run!

1. **Microtiterplate:** The break-apart snap-off strips are coated with Mumps Virus antigens. Immediately after removal of the strips, the remaining strips should be resealed in the aluminium foil along with the desiccant supplied and stored at 2 - 8°C.
2. **Washing Solution (20x conc.):** Dilute Washing Solution 1 + 19; e. g. 10 mL Washing Solution + 190 mL distilled water. The diluted buffer is stable for 5 days at room temperature (20 - 25°C). In case crystals appear in the concentrate, warm up the solution to 37°C e.g. in a water bath. Mix well before dilution.
3. **TMB Substrate Solution:** The reagent is ready to use and has to be stored at 2 - 8°C, away from the light. The solution should be colourless or could have a slight blue tinge. If the substrate turns into blue, it may have become contaminated and should be thrown away.

Assay Procedure

Please read the instruction for use carefully **before** performing the assay. Result reliability depends on strict adherence to the instruction for use as described. The following test procedure is only validated for manual procedure. If performing the test on ELISA automatic systems we recommend increasing the washing steps from three up to five and the volume of Washing Buffer from 300 µL to 350 µL to avoid washing effects. Prior to commencing the assay, the distribution and identification plan for all samples and standards/controls (duplicates recommended) should be carefully established on the plate layout. Select the required number of microtiter strips or wells and insert them into the holder.

Perform all assay steps in the order given and without any delays.

A clean, disposable tip should be used for dispensing each standard/control and sample.

Adjust the incubator to 37 ± 1°C.

1. Dispense 100 µL standards/controls and diluted samples into their respective wells. Leave well A1 for the Substrate Blank.
2. Cover wells with the foil supplied in the kit.
3. **Incubate for 1 hour ± 5 min at 37 ± 1°C.**
4. When incubation has been completed, remove the foil, aspirate the content of the wells and wash each well three times with 300 µL of Washing Buffer. Avoid overflows from the reaction wells. The interval between washing and aspiration should be > 5 sec. At the end carefully remove remaining fluid by tapping strips on tissue paper prior to the next step!

Note: Washing is important! Insufficient washing results in poor precision and false results.

5. Dispense 100 µL Conjugate into all wells except for the Substrate Blank well A1.
6. **Incubate for 30 min at room temperature (20 - 25°C).** Do not expose to direct sunlight.
7. Repeat step 4.
8. Dispense 100 µL TMB Substrate Solution into all wells.
9. **Incubate for exactly 15 min at room temperature (20 - 25°C) in the dark.** A blue colour occurs due to an enzymatic reaction.
10. Dispense 100 µL Stop Solution into all wells in the same order and at the same rate as for the TMB Substrate Solution, thereby a colour change from blue to yellow occurs.

11. Measure the absorbance at 450/620 nm within 30 min after addition of the Stop Solution.

Measurement

Adjust the ELISA Microtiterplate reader **to zero** using the **Substrate Blank**.

If - due to technical reasons - the ELISA Microtiterplate reader cannot be adjusted to zero using the Substrate Blank, subtract its absorbance value from all other absorbance values measured in order to obtain reliable results!

Measure the absorbance of all wells at **450 nm** and record the absorbance values for each standard/control and sample in the plate layout.

Bichromatic measurement using a reference wavelength of 620 nm is recommended.

Where applicable calculate the mean absorbance values of all duplicates.

Quality Control

In order for an assay run to be considered valid, these Instructions for Use have to be strictly followed and the following criteria must be met:

Substrate Blank: Absorbance value < **0.100**

Negative Control: Absorbance value < **0.200** and < **Cut-off**

Cut-off Control: Absorbance value **0.150 – 1.300**

Positive Control: Absorbance value > **Cut-off**

If these criteria are not met, the test is not valid and must be repeated.

Calculation

The Cut-off is the mean absorbance value of the Cut-off Control determinations.

Example: Absorbance value Cut-off Control 0.44 + absorbance value Cut-off control 0.42 = 0.86 / 2 = 0.43

Cut-off = 0.43

Results in Units [U]

$$\frac{\text{Sample (mean) absorbance value} \times 10}{\text{Cut-off}} = [\text{Units} = \text{U}]$$

Example:
$$\frac{1.591 \times 10}{0.43} = 37 \text{ U (Units)}$$

Interpretation Of Results

Cut-off: 10 U

Positive: > 11 U--Antibodies against the pathogen are present. There has been a contact with the antigen (pathogen resp. vaccine).

Equivocal: 9 – 11 U--Antibodies against the pathogen could not be detected clearly. It is recommended to repeat the test with a fresh sample in 2 to 4 weeks. If the result is equivocal again the sample is judged as **negative**.

Negative: < 9 U--The sample contains no antibodies against the pathogen. A previous contact with the antigen (pathogen resp. vaccine) is unlikely

Performance Characteristics

The diagnostic specificity is defined as the probability of the assay of scoring negative in the absence of the specific analyte. It is 97.68% (95% confidence interval: 95.03% - 99.15%).

The diagnostic sensitivity is defined as the probability of the assay of scoring positive in the presence of the specific analyte. It is 94.44% (95% confidence interval: 72.71% - 99.86%).

Precision

The results refer to the groups of samples investigated; these are not guaranteed specifications.

Intraassay	n	Mean (E)	CV (%)
#1	24	0.600	6.05
#2	24	1.039	4.81
#3	24	0.639	3.71
Interassay	n	Mean (U)	CV (%)
#1	12	21.38	7.50
#2	12	13.88	8.95
#3	12	1.39	10.35

Specificity

Investigation of a sample panel with antibody activities to potentially cross-reacting parameters did not reveal evidence of false-positive results due to cross-reactions.

Interferences

Interferences with hemolytic, lipemic or icteric samples are not observed up to a concentration of 10 mg/mL hemoglobin, 5 mg/mL triglycerides and 0.5 mg/mL bilirubin.

Precautions

1. The test procedure, the information, the precautions and warnings in the instructions for use have to be strictly followed. The use of the testkits with analyzers and similar equipment has to be validated. Any change in design, composition and test procedure as well as for any use in combination with other products not approved by the manufacturer is not authorized; the user himself is responsible for such changes. The manufacturer is not liable for false results and incidents for these reasons. The manufacturer is not liable for any results by visual analysis of the patient samples.
2. Only for research use.
3. All materials of human or animal origin should be regarded and handled as potentially infectious.
4. All components of human origin used for the production of these reagents have been tested for anti-HIV antibodies, anti-HCV antibodies and HBsAg and have been found to be non-reactive.
5. Do not interchange reagents or Microtiterplates of different production lots.
6. No reagents of other manufacturers should be used along with reagents of this test kit.
7. Do not use reagents after expiry date stated on the label.



8. Use only clean pipette tips, dispensers, and lab ware.
9. Do not interchange screw caps of reagent vials to avoid cross-contamination.
10. Close reagent vials tightly immediately after use to avoid evaporation and microbial contamination.
11. After first opening and subsequent storage check conjugate and standard/control vials for microbial contamination prior to further use.
12. To avoid cross-contamination and falsely elevated results pipette patient samples and dispense reagents without splashing accurately into the wells.
13. The ELISA is only designed for qualified personnel following the standards of good laboratory practice (GLP).
14. For further internal quality control each laboratory should additionally use known samples.
15. Residues of chemicals and preparations are generally considered as hazardous waste. The disposal of this kind of waste is regulated through national and regional laws and regulations. Contact your local authorities or waste management companies which will give advice on how to dispose hazardous waste.

Limitations

Bacterial contamination or repeated freeze-thaw cycles of the sample may affect the absorbance values.

References

1. Rubulavirus. Mumpsvirus (2009). In Herbert Hof, Rüdiger Dörries, Gernot Geginat: Medizinische Mikrobiologie. [Immunologie, Virologie, Bakteriologie, Mykologie, Parasitologie, klinische Infektiologie, Hygiene] ; 237 Tabellen. 4., vollst. überarb. und erw. Aufl. Stuttgart: Thieme (Duale Reihe), pp. 224–225.
2. Berbers, Guy A.; Marzec, Antonio H. J. O.; Bastmeijer, Margot; van Gageldonk, Pieter G.; Plantinga, André D. (1993): Blocking ELISA for detection of mumps virus antibodies in human sera. In Journal of virological methods 42 (2-3), pp. 155–168.
3. Berger, R.; Just, M. (1980): Comparison of five different tests for mumps antibodies. In Infection 8 (5), pp. 180–183. DOI: 10.1007/BF01639028.
4. Bienz, Kurt A. (2005): Viruses as Human Pathogen. In Fritz H. Kayser, Kurt A. Bienz, Johannes Eckert, Rolf M. Zinkernagel: Medical microbiology. Stuttgart, New York: Thieme (Thieme Flexibook), pp. 412–474.
5. Forsey, Timothy; Bentley, Maureen L.; Minor, Philip D.; Begg, Norman (1992): Mumps vaccines and meningitis. In Lancet (London, England) 340 (8825), p. 980.
6. Friedman, M.; Hadari, I.; Goldstein, V.; Sarov, I. (1983): Virus-specific secretory IgA antibodies as a means of rapid diagnosis of measles and mumps infection. In Israel journal of medical sciences 19 (10), pp. 881–884.
7. Gerike, E.; Tischer, A. (1993): Erfahrungen mit der Schutzimpfung gegen Masern, Mumps und Roteln in den neuen Bundesländern. In Gesundheitswesen (Bundesverband der Ärzte des Öffentlichen Gesundheitsdienstes (Germany)) 55 (1), pp. 38–39.